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info@mobee.eu • www.mobee.eu

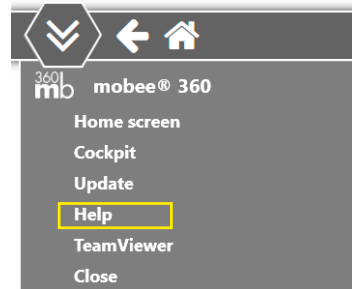
System requirements

This booklet is limited to the information and steps required for the commissioning of the different mobee® 360 systems.

You can access the detailed user manuals in the my mobee® Business Portal via the “Help” menu item.

e.g. Products > mobee® fit > Manage > Downloads > Documentation

System requirements

For trouble-free use of the software, you need a **Windows® compatible PC**. Your system should meet the following **minimum requirements**:

- Processor performance/CPU min. IntelCore i3 of a current processor generation or respective AMD equivalent (x64-based processor)
Important note: ARM processors are not supported
- RAM (= random access memory) min. 8 GB
- Display resolution min. Full HD (1920 x 1080)
- 5 GB of available hard disk space
- free USB slots depending on device type and quantity
- PDF Reader for displaying printouts and digital user documentation
- Operating system: from Windows 10
- Integrated Bluetooth: BLE (Bluetooth Low Energy) version 4.2 and higher, recommended for all devices with Bluetooth communication

Installation

- Software installers are available in the my mobee® Business Portal *e.g. Products > mobee® fit > Manage > Software*
- Make sure that you have sufficient administrator rights for the PC on which you want to install the software!

Activation

You can obtain the license keys required for activation in the my mobee® Business Portal

Help

All service rates include a one-time online start-up assistance. Arrange your appointment via our website or the calendar link sent to you.

You will find **starter webinars** (german) on various topics after registration in the my mobee® Business Portal.

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Generell safety instructions – keep away from:

- Damaged cables or plugs
- Devices with damaged casing
- Own attempts at battery change
(on devices with permanently installed battery)

In these cases, please contact your distributor!

For a properly functioning and durable system

- Avoid dropping or hitting the device.
- Disinfection according to device-specific user documentation
- Observe the transport, storage and use instructions in the respective product documentation.

Contents – check completeness – start up

In case of transport damage or missing components, please contact your distributor immediately.

Note

The actual appearance of the products may vary depending on product variants and/or time of delivery may differ from the illustrations shown here. These are for orientation purposes only.

mobee® fit / mobee® med / mobee® care

- 1 mobee® device
- 3 elastic fastening straps & one strap clip each
- 1 USB radio stick for wireless communication (mobee com)
- 1 charging cable USB to micro USB
- 1 power plug 230V with USB connection
(this accessory may be omitted depending on the country)
- 1 USB extension cable 2.0 A-A, 180cm
- 1 transport case

Fully charge the battery of the mobee® device with the supplied USB charging system before you start up the system.

To plug in the charging cable, carefully open the rubber plug at the bottom edge of the device.

The battery is fully charged after about 4-6 hours. You can recognize the completed charging process by the fact that both LEDs of the device light up continuously.

Remove the charging cable from the device. It is now ready for use and can be switched on and connected in the software. To switch the device on and off, press and hold the center button for 3 seconds.

The device must first be connected in the relevant software. Follow the instructions in the corresponding user manual.

Stress Pilot® Pro

- 1 Stress Pilot® Pro device
- 1 ear clip / sensor
- 1 Polar H10 sensor with chest strap
- 1 USB to micro USB cable
- 1 transport case

The device is connected to the computer via USB (or Bluetooth in case of PolarH10) and is then ready for use. In the further course, follow the instructions in the corresponding user manual.

mobee® spine

- 1 mobee® spine device
- 1 transport case
- 1 charging/data cable (USB-C > USB-C)

The device communicates via Bluetooth and must be initially connected in the software. In the further course, follow the instructions in the corresponding user manual.

S3 CHECK Pro

- 1 S3 CHECK Pro balance plate incl. sensor
- Safety rail if purchased
- 1 USB connection cable
- carrying bag

The device is connected to the computer via USB and is then ready for use. ready for use. In the further course, follow the instructions in the corresponding user manual.

TANITA scale

- 1 TANITA scale (with or without stand)
- 1 USB connection cable
- transport case (optional accessory)

The device is connected to the computer via USB and has to be connected in the software. In the further course, follow the instructions in the corresponding user manual.

mobee® metabolic

- 1 mobee® metalyzer3B
- 1x O2 sensor
- 1x sample line, approx. 2m
- 1x Nano Bluetooth USB adapter
- 1x CORTEX Face mask set incl. 2x adapter
- 1x hair net for face masks
- 1x med. table power supply
- 1x power cable Europe
- 1x gas-on-demand valve
- 120 disposable turbines, alternatively reusable turbine (optional variant)

For the commissioning of mobee® metabolic you will receive a separate manual. Please be sure to follow the steps described in the steps described there.

Use of the mobee® 360 products

Below are our recommended steps that should ideally be performed before each deployment:

- Check for pending Windows updates
- Check for available updates of our software modules
- Internet connection required for updates
- complete charging (depending on the system) and successful connection
- short function test.

Persons working with the systems and software, should have received prior instruction in the correct method of operation (preparation, use, interpretation, etc.).

Information on training, webinars or workshops can be found on our website and in the my mobee® Business Portal.

Support

If you need help, please contact our support team.
Prerequisite: valid service contract.

- Submit a support request in the my mobee® Business Portal or our website
- A ticket will be opened automatically in our support system.
- We will get back to you promptly by phone.

Support hours (CET)

Mon.-Fri. 9-12 a.m. and 1-4 p.m.
(except lux. holidays and company closing days)

Contact

SportMed SA – Headquarters
46, route de Wasserbillig
L-6490 Echternach

SportMed SA – Sales Office
91, Esplanade de la Moselle
L-6637 Wasserbillig

DE tel +49 6561 69771-72
LU tel +352 26720101

info@mobee.eu • www.mobee.eu

SportMed_{SA}

Version 6.2
gültig ab November 2024 / valid as of November 2024